GE Healthcare

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## REFERANSLAR

1. Benacerraf, B.R., T.D. Shipp, and B.Bromley, Improving the efficiency of gynecologic sonography with 3-dimensional volumes: a pilot study. *J Ultrasound Med*, 2006. 25(2): p. 165-71.
2. Chaoui, R., J.M. Levailant, B. Benoit, C. Faro, P. Wegrzyn, and K.H. Nicolaides, Three-dimensional sonographic description of abnormal metopic suture in second- and third-trimester fetuses. *Ultrasound Obstet Gynecol*, 2005. 26(7): p. 761-4.
3. Ghi, T., P. Casadio, M. Kuleva, A.M. Perrone, L. Savelli, S. Giunchi, M.C. Meriggiola, G. Gubbini, G. Pilu, C. Pelusi, and G. Pelusi, Accuracy of three-dimensional ultrasound in diagnosis and classification of congenital uterine anomalies. *Fertil Steril*, 2009. 92(2): p. 808-13.
4. Jurkovic, D., A. Geipel, K. Gruboeck, E. Jauniaux, M. Natucci, and S. Campbell, Three-dimensional ultrasound for the assessment of uterine anatomy and detection of congenital anomalies: a comparison with hysterosalpingography and two-dimensional sonography. *Ultrasound Obstet Gynecol*, 1995. 5(4): p. 233-7.
5. Jurkovic, D., K. Gruboeck, A. Tailor, and K.H. Nicolaides, Ultrasound screening for congenital uterine anomalies. *Br J Obstet Gynaecol*, 1997. 104(11): p. 1320-1.
6. Pilu, G., T. Ghi, A. Carletti, M. Segata, A. Perolo, and N. Rizzo, Three-dimensional ultrasound examination of the fetal central nervous system. *Ultrasound Obstet Gynecol*, 2007. 30(2): p. 233-45.
7. Pilu, G. and M. Segata, A novel technique for visualization of the normal and cleft fetal secondary palate: angled insonation and three-dimensional ultrasound. *Ultrasound Obstet Gynecol*, 2007. 29(2): p. 166-9.
8. Platt, L.D., G.R. Devore, and D.H. Pretorius, Improving cleft palate/cleft lip antenatal diagnosis by 3-dimensional sonography: the "flipped face" view. *J Ultrasound Med*, 2006. 25(11): p. 1423-30.
9. Ten, P.M., J.P. Pedregosa, B. Santacruz, B. Adiego, E. Barron, and W. Sepulveda, Three-dimensional ultrasound diagnosis of cleft palate: "reverse face", "flipped face" or "oblique face" –which method is best? *Ultrasound Obstet Gynecol*, 2009. 33(4): p. 399-406.
10. Benacerraf, B.R., T.D. Shipp, and B. Bromley, Which patients benefit from a 3D reconstructed coronal view of the uterus added to standard routine 2D pelvic sonography? *AJR Am J Roentgenol*, 2008. 190(3): p. 626-9.

11. Benacerraf, B.R., T.D. Shipp, and B. Bromley. Three-dimensional ultrasound detection of abnormally located intrauterine contraceptive devices which are a source of pelvic pain and abnormal bleeding. *Ultrasound Obstet Gynecol*, 2009. 34(1): p. 110-5.

#### EUROPE

GE Ultraschall Deutschland GmbH  
Beethovenstr. 239  
D-42655 Solingen  
T 49 212-28 02-0  
F 49 212-28 02 28

#### UNITED KINGDOM

GE Medical Systems Ultrasound  
2, Napier Road  
Bedford MK41 0JW  
Phone: (+44) 1234 340881  
Fax: (+44) 1234 266261

#### AMERICAS

GE Medical Systems  
Milwaukee, WI, USA  
Fax: (+1) 262 544-3384

#### ASIA

GE Medical Systems  
Tokyo, Japan  
Fax: (+81) 3-3223-8524  
Shanghai, China  
Fax: (+86) 21-5208 0582

Visit us online at:  
[www.gehealthcare.com](http://www.gehealthcare.com)

© 2009 General Electric Company – All rights reserved.  
GE Healthcare, a division of General Electric Company.

General Electric Company reserves the right to make changes in specifications and features shown herein, or discontinue the product described at any time without notice or obligation. Contact your GE representative for the most current information.

General Electric Company, doing business as GE Healthcare.

GE, GE Monogram, Voluson®, LOGIQ® and CrossXBeamCRI are trademarks of General Electric Company.

GE imagination at work